

## SPECIMEN REQUIREMENTS – BLADDER TURBT TISSUE

### SPECIMEN AND BLOCK SELECTION

1. Verify tumor pathology report indicates definitive diagnosis of muscle-invasive urothelial carcinoma (MIBC).
2. Select TURBT specimen with invasive tumor used to establish MIBC diagnosis.
3. Select H&E slide and FFPE block most representative of the invasive tumor.

### SAMPLING INSTRUCTIONS

Please select one of the following options:

#### OPTION 1: BLOCK

1. Submit the selected FFPE block.

#### OPTION 2: SLIDES

1. Submit one (1) H&E slide and nine (9) 5µm thick serially sectioned unstained slides or ten (10) 5µm thick serially sectioned unstained slides.
2. Label the serial sectioned slides (1-10) in the order in which they were cut.
3. Store unstained slides at 4 degrees C until shipping.

## SHIPPING REQUIREMENTS

### BLOCK

1. Place a barcoded label on the block, then place the selected FFPE block into the small zip-lock bag.
2. Place the small zip-lock bag containing the block into the foam insert.

### SLIDES

1. Place slides into the two (2) slide holders (up to 5 slides each).
2. Apply barcoded labels to the two (2) slide holder(s) and place the labeled slide holder(s) back into the foam insert in the specimen kit.

---

3. Apply barcoded labels to the test requisition and pathology report.
4. Fold (a) Test Requisition and (b) Pathology Report and place all paperwork into the specimen kit, between the dividing flap and the foam insert.
5. Fold down the dividing flap, place the provided cold pack on top of the flap and close the specimen kit.
6. Place the closed specimen kit into the FedEx® Clinical Lab Pak and apply the shipping label provided.
7. Ship via your regular scheduled FedEx® pickup or call Customer Support @ 888.792.1601 to schedule a pick up.

IF YOU HAVE ANY QUESTIONS ABOUT SPECIMEN SUBMISSION, PLEASE CONTACT OUR CUSTOMER SUPPORT TEAM AT 888.792.1601 OR EMAIL [CS@DECIPHERBIO.COM](mailto:CS@DECIPHERBIO.COM)

Testing is performed by Decipher Corp., a Decipher Biosciences company

LAB-FRM-10007 v2.0  
© 2019 DECIPHER BIOSCIENCES, INC.

DECIPHERBIOSCIENCES  
6925 LUSK BOULEVARD, SUITE 200, SAN DIEGO, CA 92121-2789  
T 888.792.1601 | F 855.324.2768 | [DECIPHERBIO.COM](http://DECIPHERBIO.COM)

