

Frequently Asked Questions

Who orders Decipher Prostate?

Decipher Prostate must be ordered by your doctor or provider. No additional procedure is required.

How long does it take to get my Decipher Prostate test results?

Your Decipher Prostate test results will be provided to your doctor approximately 2-3 weeks after the date it was ordered.

Does Decipher Prostate test my DNA?

No, Decipher Prostate evaluates gene expression by measuring the tumor's RNA. This provides a glimpse of the tumor's activity or behavior.

What is tumor genomic testing?

Tumor genomic testing looks at genes, how they are interrelated, and their expression or activity within the cancerous tissue. This can help determine the behavior of the disease, such as its aggressiveness.

What information is used to determine my Decipher score?

Your Decipher score is derived from the expression of 22 genes from your prostate cancer tumor, and does not consider any other clinical information such as Gleason score or PSA level.

Guidelines & Coverage

- Included in the NCCN Clinical Practice Guidelines In Oncology (**NCCN Guidelines®**) & **AUA/ASTRO Guidelines** for Prostate Cancer
- Covered by **Medicare** for all beneficiaries with localized prostate cancer & **major insurance plans** throughout the US
- Financial assistance is available via the **Veracyte Access** program.

Download application form using QR code



| Household Size | Cost to Patient | | |
|--------------------------------|-----------------|-----------|-----------|
| | \$0 | \$100 | \$295 |
| 1 | \$30,120 | \$45,180 | \$60,240 |
| 2 | \$40,880 | \$61,320 | \$81,760 |
| 3 | \$51,640 | \$77,460 | \$103,280 |
| 4 | \$62,400 | \$93,600 | \$124,800 |
| 5 | \$73,160 | \$109,740 | \$146,320 |
| 2024 maximum household income* | | | |

NCCN = National Comprehensive Cancer Network® (NCCN®). Referenced with permission from the NCCN Clinical Practice Guidelines in Oncology (NCCN Guidelines®) for Prostate Cancer V.3.2024. © National Comprehensive Cancer Network, Inc. 2024. All rights reserved. Accessed March 22, 2024. To view the most recent and complete version of the guideline, go online to NCCN.org. NCCN makes no warranties of any kind whatsoever regarding their content, use or application and disclaims any responsibility for their application or use in any way.


*Veracyte Access eligibility is based on total annual household income, insurance status, and household size. Some eligibility restrictions apply. Veracyte offers certain Decipher tests at a reduced cost to eligible applicants except where otherwise restricted. Residents of the United States, District of Columbia and Puerto Rico are eligible to apply. Veracyte Access does not constitute health insurance. You must meet certain income requirements set forth above. We may request documentation to verify your income, including recently filed tax returns and other supporting documentation. By requesting assistance, you certify, to the best of your knowledge, that you are eligible for assistance and that you have insufficient financial resources to pay for the ordered test. We may discontinue or change this program at any time for any reason without notice.

© 2024 Veracyte, Inc. All rights reserved. Decipher, Veracyte, and the Veracyte logo are registered trademarks of Veracyte. Decipher Prostate is available in the United States as part of Veracyte's CLIA-Validated laboratory developed test (LDT) service, and FDA clearance is not required.

MRKT-FRM-10040-E



Decipher® Prostate Genomic Classifier

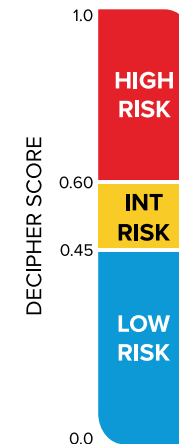

Talk to your doctor about how Decipher Prostate may help to inform important decisions for your prostate cancer care.

Contact Us!

client.service-urology@veracyte.com

1.888.792.1601

<https://decipherbio.com/decipher-prostate/patients/decipher-prostate-overview/> OR


Scan QR Code



About Decipher Prostate

- It is a 22-gene genomic test that looks at the molecular characteristics of your tumor to assess its **aggressiveness**, or likelihood of spreading outside the prostate.
- The result is a number from 0-1 (your Decipher score), that is classified as **Decipher Low Risk**, **Decipher Intermediate Risk** or **Decipher High Risk**.
- Testing is performed on existing prostate tumor tissue from your biopsy or radical prostatectomy (surgery to remove the prostate).

No additional procedure is required.

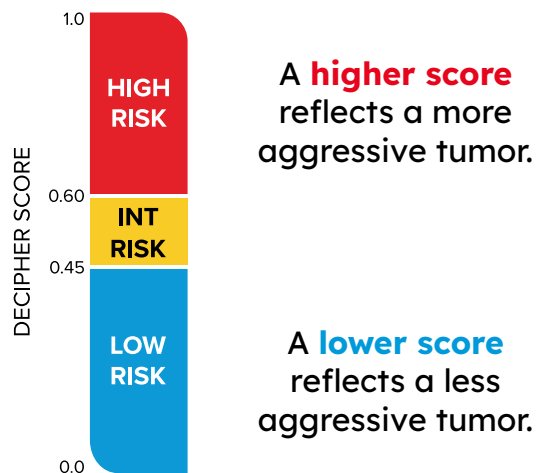

Why Did My Doctor Order Decipher Prostate?

Before you & your doctor decide how to manage your prostate cancer, your doctor will assess your **risk level** by considering all available information about your tumor, including your:



- » Gleason score,
- » Prostate specific antigen (PSA) level &
- » Clinical stage.

Based on these factors alone, it is not always clear **when** or **if** treatment is needed and, if so, **how much** treatment is necessary.



More information about the biology of your tumor (i.e., your **Decipher score**) can help determine the best management plan for you.

How Will My Decipher Prostate Test Results be Used?

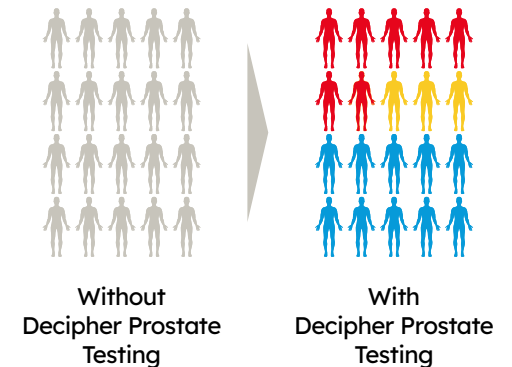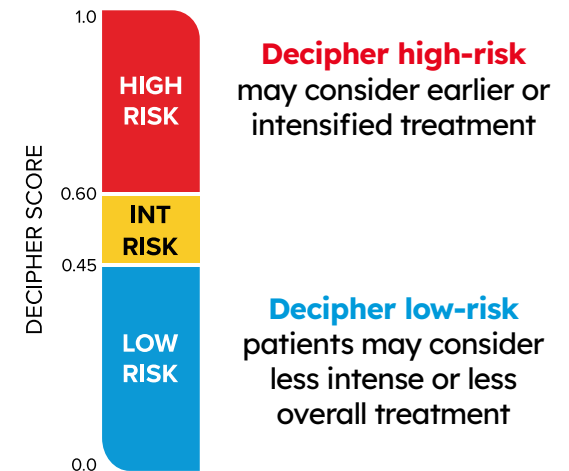

Talk to your doctor about how Decipher Prostate may help to inform important decisions for your prostate cancer care.