

Decipher® Prostate

Genomic Classifier

Specimen Requirements - Biopsy and Radical Prostatectomy Tissue

PLEASE SELECT ONE OF THE FOLLOWING OPTIONS BELOW WHEN SUBMITTING SPECIMENS FOR DECIPHER TESTING

OPTION 1: BLOCK SUBMISSION

For Biopsy:

Select one (1) formalin fixed paraffin embedded (FFPE) block of diagnostic needle core biopsy with at least 0.5 mm in total linear length of tumor tissue with the highest Grade Group (Gleason score) of tumor.

For Radical Prostatectomy:

Select one (1) formalin fixed paraffin embedded (FFPE) tissue block with the highest Grade Group (Gleason score) and at least 0.5 cm² of tumor tissue by area.

OPTION 2: SLIDE SUBMISSION

For Biopsy:

Submit ten (10) 5µm thick serially sectioned unstained slides of tumor tissue with the highest Grade Group (Gleason score) of tumor. Label the serial sectioned slides (1-10) in the order in which they were cut. Air dry slides overnight or until no water is visible under the paraffin section and store unstained slides at 4 degrees C until shipping.

For Radical Prostatectomy:

Submit six (6) 5µm thick serially sectioned unstained slides of tumor tissue with the highest Grade Group (Gleason score) of tumor. Label the serial sectioned slides (1-6) in the order in which they were cut. Air dry slides overnight or until no water is visible under the paraffin section and store unstained slides at 4 degrees C until shipping.

SHIPPING REQUIREMENTS

OPTION 1: BLOCK

1. Place the selected FFPE block into the small zip-lock bag.
2. Place the small zip-lock bag containing the block into the foam insert.

OPTION 2: SLIDES

1. Place slides into the two (2) slide holders (up to 5 slides each).
2. Place the slide holder(s) back into the foam insert in the specimen kit.
3. Fold Test Requisition, Pathology Report and place all paperwork into the specimen kit, between the dividing flap and the foam insert.
4. Fold down the dividing flap, place the provided cold pack on top of the flap and close the specimen kit.
5. Place the closed specimen kit into the FedEx® Clinical Lab Pak and apply the shipping label provided.
6. Ship via your regular scheduled FedEx® pickup or call Customer Support @ 1.888.792.1601 to schedule a pick up.

IF YOU HAVE ANY QUESTIONS ABOUT SPECIMEN SUBMISSION, PLEASE CONTACT OUR CUSTOMER SUPPORT TEAM AT 1.888.792.1601 OR EMAIL CLIENT.SERVICE-UROLOGY@VERACYTE.COM



Veracyte Labs SD

6925 Lusk Boulevard, Suite 200
San Diego, CA 92121

T 1.888.792.1601
F 1.858.766.6575

E client.service-urology@veracyte.com
W veracyte.com/decipher